Mölnlycke®

# Mepilex® Transfer

WITH SAFETAC® TECHNOLOGY

Manufacturer / Fabricant

 Mölnlycke Health Care AB  
Gamlestadvägen 3C, Box 13080  
SE-402 52 Göteborg, Sweden

[www.molnlycke.com](http://www.molnlycke.com)

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Soft silicone exudate transfer dressing

## Product description

Mepilex Transfer is a highly conformable dressing that transfers exudate and, in combination with appropriate secondary dressing, maintains a moist wound environment and minimises the risk for maceration. The dressing has a Safetac® wound contact layer that is a unique adhesive technology.

It minimises pain to patients and trauma to wounds and the surrounding skin at dressing removal.

Mepilex Transfer consists of

- a soft silicone wound contact layer (Safetac)
- a thin, flexible absorbent pad of polyurethane foam

Dressing material content:

Silicone, polyurethane

## Indications for use

Mepilex Transfer is intended for a wide range of exuding wounds.

Mepilex Transfer can also be used as protection of compromised and/or fragile skin.

## Precaution(s)

- Do not use on patients with known hypersensitivity to the dressing or its materials.
- Do not use together with oxidising agents such as hypochlorite solutions or hydrogen peroxide.
- If you see signs of clinical infection e.g. fever or if the wound or surrounding skin becomes red, warm or swollen, consult a health care professional for appropriate treatment.
- Do not reuse, if reused performance of the product may deteriorate and cross contamination may occur.
- Sterile. Do not use if sterile barrier is damaged or opened prior to use. Do not re-sterilise.

## Instructions for use

Mepilex Transfer can be used by laypersons under the supervision of health care professionals.

1. Cleanse the wound in accordance with clinical practice.
2. Dry the surrounding skin thoroughly.
3. Select an appropriate dressing size. The dressing should cover the dry surrounding skin by at least 1-2 cm for the smaller sizes (sizes up to 12.5x12.5 cm) and 3-5 cm for the larger sizes. If required, the dressing may be cut to suit various wound shapes and locations.
4. Remove the first release film and apply the adherent side to the wound.
5. Remove the remaining release film and smooth down the dressing on the skin. Do not stretch the dressing.
6. Apply an appropriate secondary dressing and fixation, if needed, to maintain a moist wound environment:
  - For moderately to highly exuding wounds, use a secondary absorbent dressing on top of Mepilex Transfer and fixate them with e.g. Tubifast®, hosiery or bandaging
  - For non-exuding to low exuding wounds or for protection of fragile skin, use e.g. a breathable film dressing or Tubifast for fixation.

The dressing change interval for Mepilex Transfer may be several days. Change the dressing before it is fully saturated, at signs of leakage or as indicated by clinical practice.

When an absorbent secondary dressing is used, this dressing may need to be changed more frequently than the Mepilex Transfer dressing underneath it. Change the secondary dressing before it is fully saturated, at signs of leakage or as indicated by clinical practice. When changing the secondary dressing, clean the surrounding skin and inspect the wound on a regular basis.

When Mepilex Transfer is used for protection, inspect the skin underneath on a regular basis. Mepilex Transfer can be used under compression bandaging and in combination with gels. Disposal should be handled according to local environmental procedures.

## Other information

The polyurethane foam used in the product may change colour to more yellow when it is exposed to light, air and/or heat. The colour change has no influence on product properties when used before expiry date.

If any serious incident has occurred in relation to the use of Mepilex Transfer, it should be reported to Mölnlycke Health Care.

Mepilex®, Safetac® and Tubifast® are registered trademarks of Mölnlycke Health Care AB.

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